
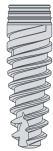


RECOMMENDED DRILLING PROTOCOL

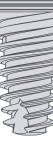


Surgical Protocol

								
Implant	Bone	Ø 2.0 PD200L16C	Ø 2.8 SD2028L16C	Ø 3.2 SD2832L16C	Ø 3.6 SD3236L16C	Ø 4.2 SD3642L16C	Ø 4.6 SD4246L16C	Ø 5.0 CD 50
 Ø 3.3	Soft D4	1 	1/4 	(1/4 )	(1/4 )			
	D2-3	1 	1 	1/2 	1/4 			
	Hard D1	1 	1 	3/4 	1/2 			
 Ø 3.75	Soft D4	1 	1/4 	(1/4 )	(1/4 )			
	D2-3	1 	1 	1 	1/2 			
	Hard D1	1 	1 	1 	1/2 			
 Ø 4.2	Soft D4	1 	1 	(1/4 )	(1/4 )	(1/4 )		
	D2-3	1 	1 	1 	1/2 	1/2 		
	Hard D1	1 	1 	1 	1 	1/2 		
 Ø 5.0	Soft D4	1 	1 	1 	(1/4 )			1 
	D2-3	1 	1 	1 	1 	1 	1 	1 
	Hard D1	1 	1 	1 	1 	1 	1 	1  *

Torque: **Max 50 Ncm** • (x): **Cortical Bone** • Drill: **The number in the brackets (x) denotes the drilling depth relative to the implant length**

* Drill SD4650L16C
Not a part of the Kit

Disclaimer: The drilling protocol provided is only a general guideline and must be adapted to the individual patient, in particular to the clinical conditions, bone quality and surgeon preference. The practitioner is responsible to determine the appropriate protocol for each case based on the practitioner's clinical knowledge, judgment and experience and the individual patient's requirements XGate Dental does not accept any liability for any inconvenience or damage resulting from the use of the provided drilling protocol information.

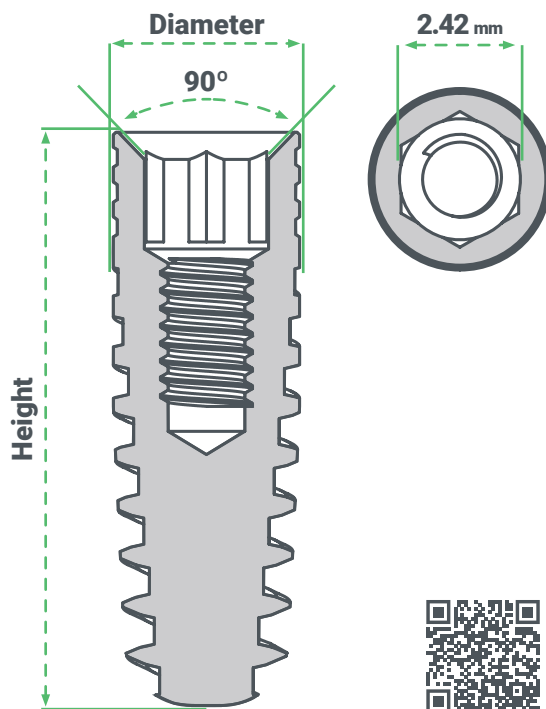

XGATE X3 Internal Hex Regular Platform



EASY KIT

**Convenient
Surgical Kit Box**



**Surgical
Protocol**



Telefon: (302) 498-8373
E-posta: info@xgate.dental

XGATE Dental Group GmbH
Falkensteiner Straße 77,
60322 Frankfurt am Main, Germany